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The screenshot shows the main interface of a device. At the top, there is a header bar with a menu icon on the left, the text "RS_ONE_21..." in the center, and a gear icon on the right. Below the header, there is a "Test Profile" section with a dropdown menu showing "001@05". Below that is a "Note:" section with a text input field containing "Admin". Below the note is a "Sample No." section with a text input field containing "004". At the bottom of the main area, there are two buttons: "Run Test" and "Set Delay". Below the buttons, there is a status bar showing "Last Test--2021/09/23-09:26:52" and "User:Admin / Battery: None".

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Run Test Analysis in __ (1-100) Minutes. Please enter a number between 1-100.
If delay time is 1 minute 30 seconds please enter 1.5.

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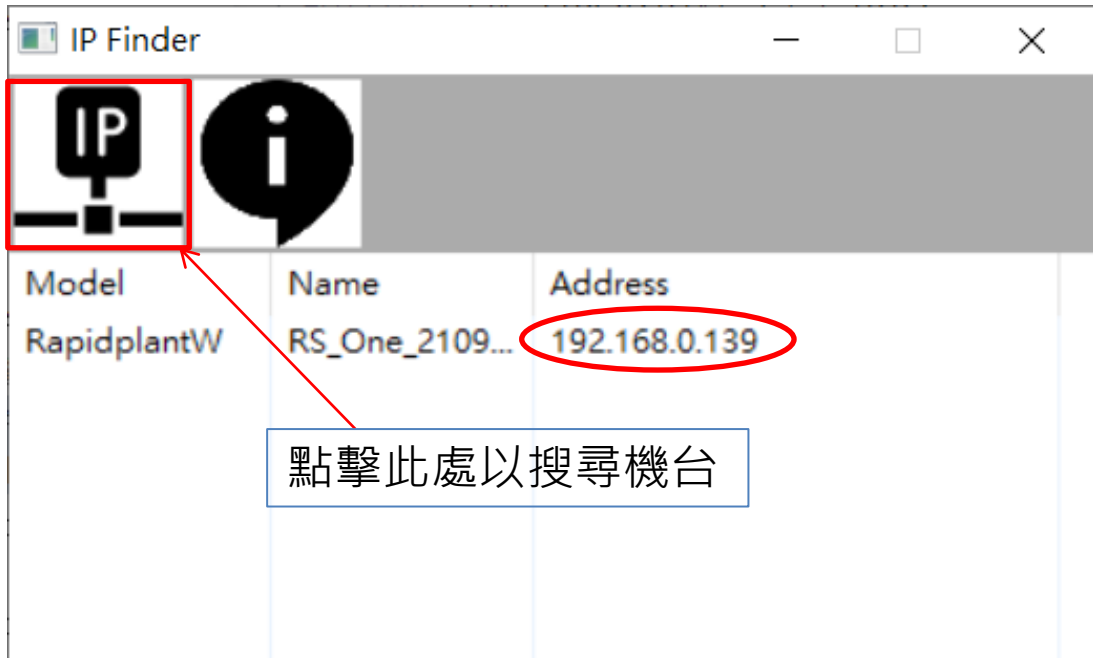
IP Finder

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
The screenshot shows the desktop application window titled "IP Finder". The window has a title bar with standard minimize, maximize, and close buttons. Below the title bar is a header area containing two icons: a computer icon with "IP" on it and an information icon. A red box highlights the computer icon, and a red arrow points from it to a text box below. Below the header is a table with three columns: "Model", "Name", and "Address". The first row of data shows "RapidplantW" in the Model column, "RS_One_2109..." in the Name column, and "192.168.0.139" in the Address column. The IP address is circled in red. A red arrow points from the computer icon to the IP address.

Model	Name	Address
RapidplantW	RS_One_2109...	192.168.0.139

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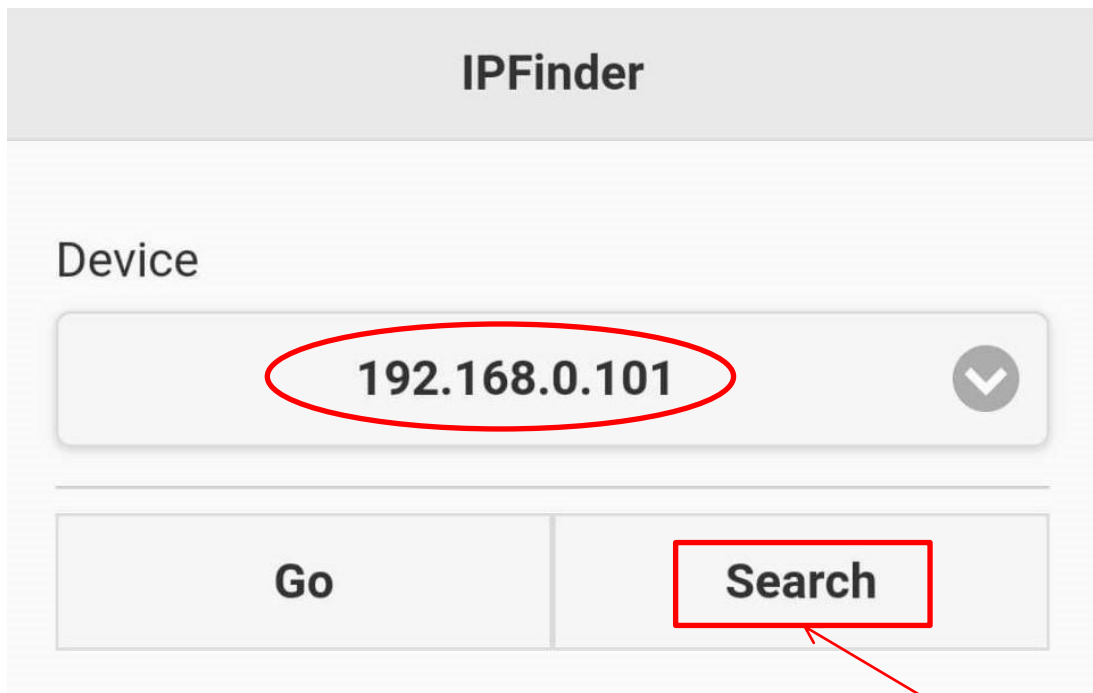The screenshot shows the mobile application interface titled "IPFinder". It features a "Device" section with a text input field containing the IP address "192.168.0.101", which is circled in red. Below the input field are two buttons: "Go" and "Search". The "Search" button is highlighted with a red box, and a red arrow points from it to a text box below.

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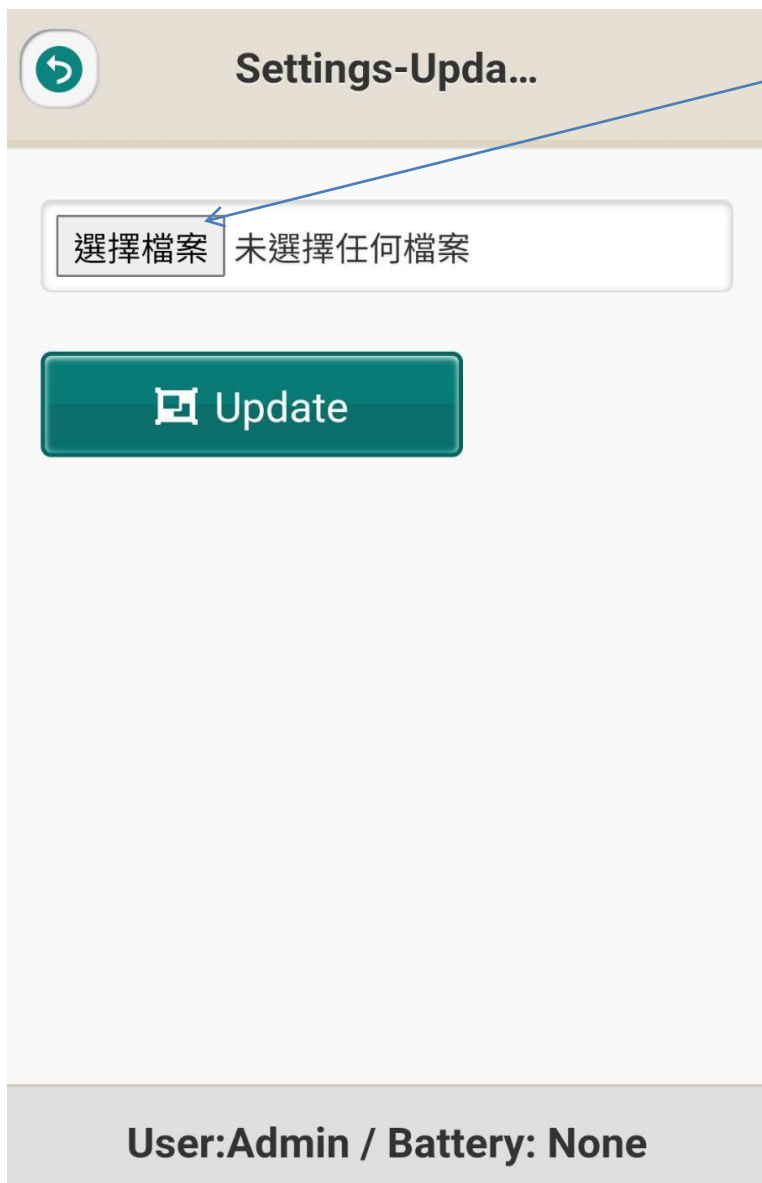
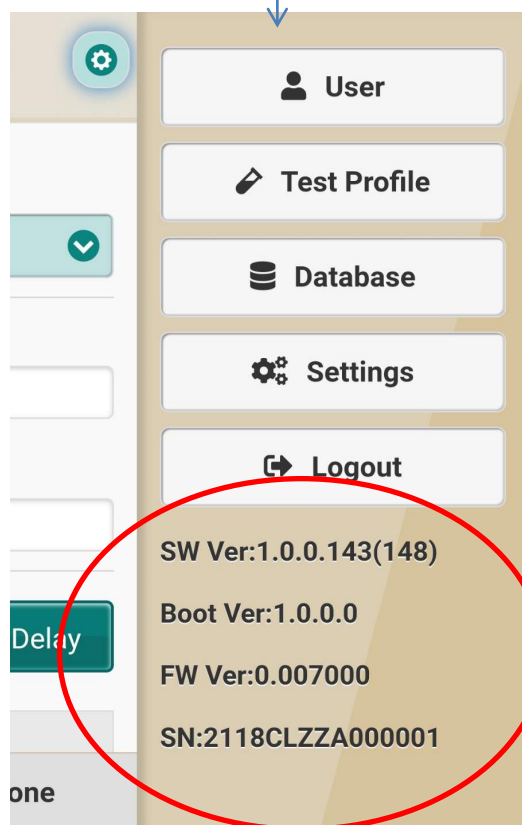
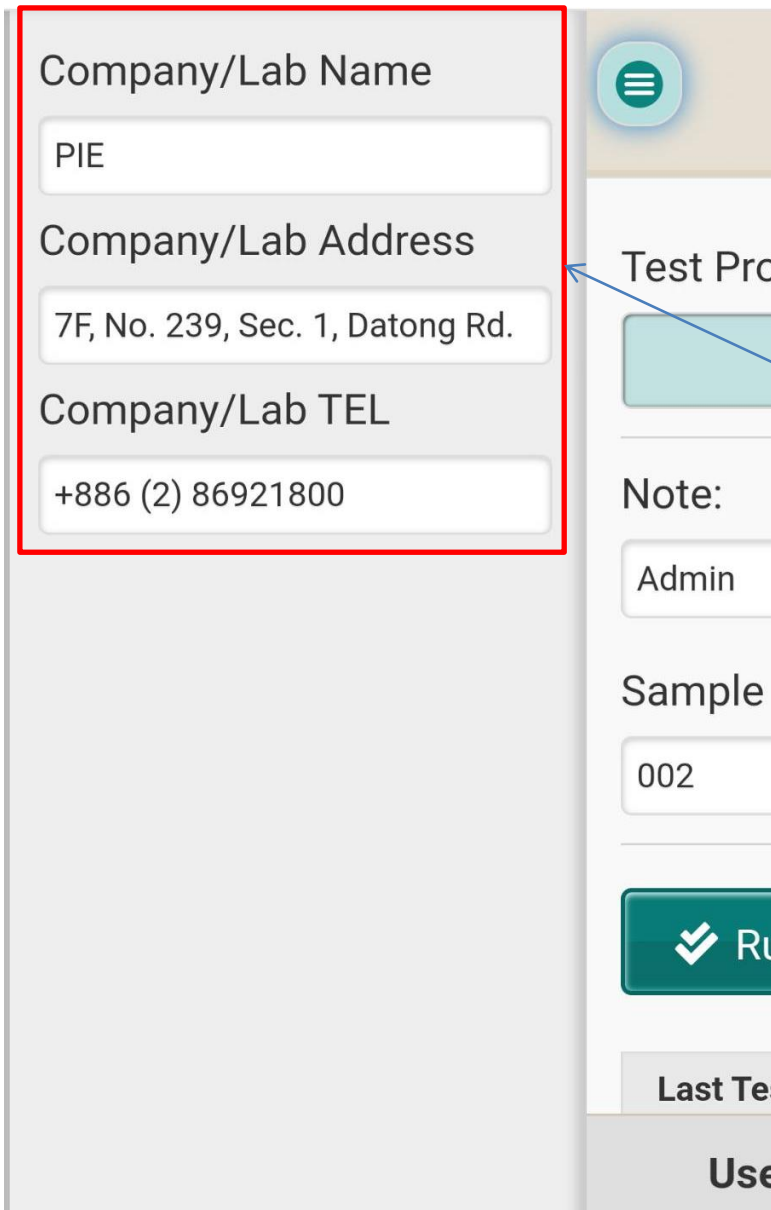
The screenshot shows a mobile application interface with a form on the left and a sidebar on the right. The form has three input fields: 'Company/Lab Name' with the value 'PIE', 'Company/Lab Address' with the value '7F, No. 239, Sec. 1, Datong Rd.', and 'Company/Lab TEL' with the value '+886 (2) 86921800'. A red rectangular box highlights these three fields. The sidebar on the right contains a menu icon at the top, followed by 'Test Pro', 'Note: Admin', 'Sample 002', a green button with a checkmark and 'Ru', and 'Last Tes' and 'Use' at the bottom.

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